



Note: NEOJECT® PLUS SAFETY needles are incompatible with other Luer or Luer-Lock syringes

## The effective protection of needlestick injuries!

### Instruction of use

**1** Draw up the medicine with standard sample needle and remove the needle afterwards


**2** Attach safety needle firmly to the safety syringe  
(Note: protective cap can only be removed after connection with the syringe)



**Inject**




**Pull the plunger backwards**

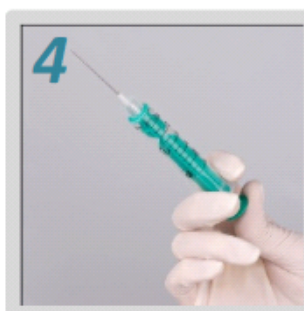

**Activate safety mechanism after injection:**

**Firmly push the plunger down**

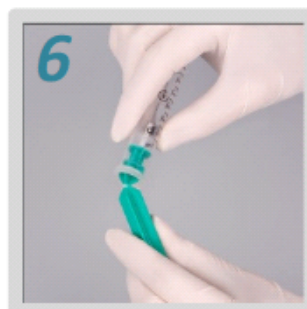

**Break off the plunger and dispose the syringe**